

- Super-absorbent dressing
- Suitable for use under all forms of compression
- Can be used as a primary or secondary dressing

Application Guide


- 1 Cleanse the wound according to local procedures
- 2 Select an appropriate dressing size. The dressing pad needs to overlap the edge of the wound. Do not cut the dressing
- 3 Open the pack and lay the sterile pad onto the wound. Either side of the sterile pad may be used. Stack dressings for additional absorption
- 4 Lightly cover with a bandage or wrap to secure the dressing, refer to your local guidelines for further guidance
- 5 Check the dressing at regular intervals and change according to the clinical condition of the wound, or when saturated

Folding Techniques


In-between Toes/Fingers




- 1 Fold a 4 x 9" in half lengthways

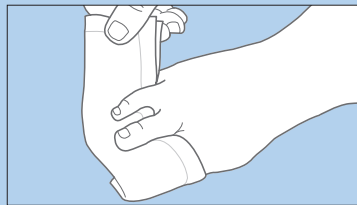


- 2 Weave in between toes/fingers

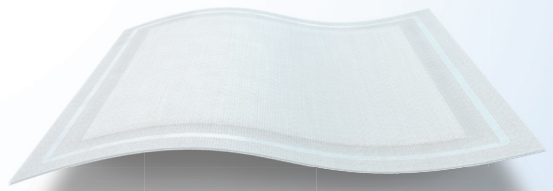


Around Peg Sites

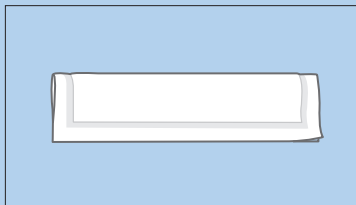

- 1 Fold a 4 x 9" in half lengthways

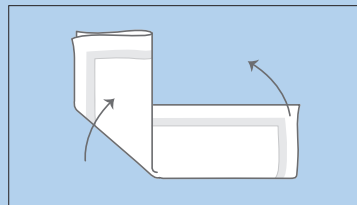


- 2 Fold the two corners over

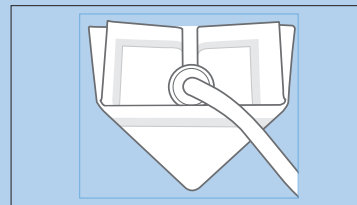


- 3 Place around the peg site


Heel/Elbow

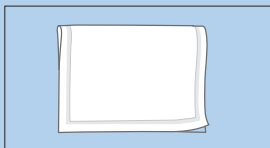


- 1 Fold a 8 x 9" in half

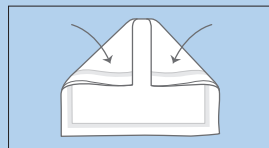


- 2 Fold each corner inwards

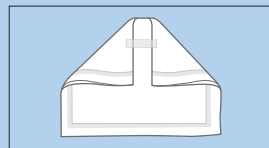

- 3 Place tape to hold folded corners in place

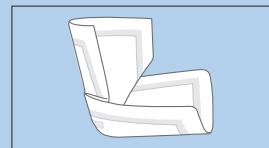


- 4 Open out folded dressing

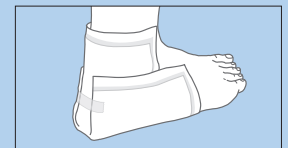

- 5 Place on heel/elbow

How to apply KerraMax Care Multisite:

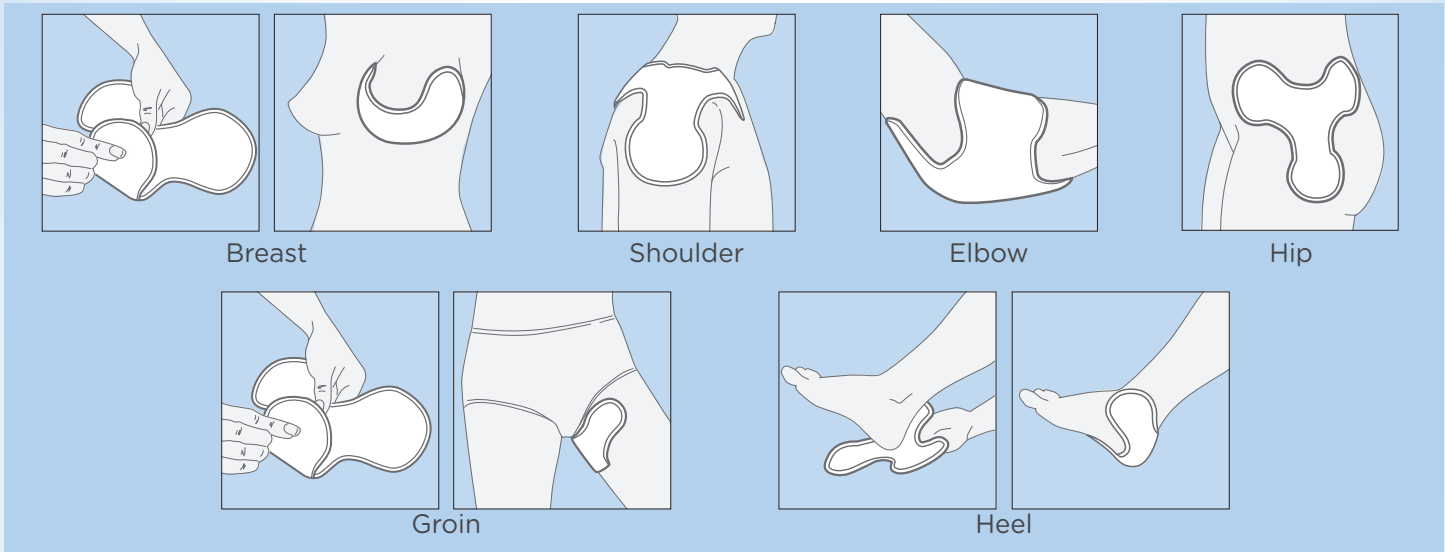

KerraMax Care® Super-absorbent dressing

KerraMax Care® Super absorbent dressing is designed to absorb and retain large amounts of exudate to minimize discomfort and dressing changes. Its wicking layer allows for increased absorption while staying dry to the touch to keep moisture away from healthy skin.

Order Information

Product	Size	Surface area, in ²	Dressings per box	HCPCS
PRD500-025	2 x 2 in	4	10	A6196
PRD500-050	4 x 4 in	16	10	A6196
PRD500-100	5 x 6 in	30	10	A6197
PRD500-120	4 x 9 in	36	10	A6197
PRD500-240	8 x 9 in	72	10	A6197
PRD500-380	8 x 12 in	96	5	A6198
PRD500-600	8 x 20 in	160	5	A6198
PRD500-300	8 x 9 in Multisite	29	5	A6197



To purchase product please contact your medical supply distributor or call 855-522-2211



Crawford Healthcare Inc. | 2005 South Easton Rd. | Suite 203 | Doylestown, PA 18901
 Tel 855-522-2211 | Email us.info@crawfordhealthcare.com
<http://us.crawfordhealthcare.com>